

**Saliva Collector for COVID-19 Validation Participants (US residents only please, sorry)
Description and How to Order**


Thank you for helping us finalize our product. As indicated in my post, we have performed all of the required FDA tests, except the most crucial test: Clinical Validation.

**We need saliva samples only from participants that will be getting a PCR test.
The Saliva Collector and a return mailer are FREE.**

If you are getting a nasal swab PCR test for work, school, sports, etc., or because you are ill (sorry), we need you to collect a saliva sample the same day you provide the nasal sample.

Four Simple Steps:

1. ***Order the Saliva Collector at least 5 days before your nasal swab test so you have it on time, at sales@rta.biz, type “Saliva Collector” in the message line.***
2. ***Collect your saliva sample the same day you provide the nasal sample.***
Hold the Sterile Sample Vial to your mouth and fill with *liquid* saliva (not bubbles) to the 2 mL mark. It doesn't have to be exact. Sucking the checks helps generate saliva. Take your time.
3. ***Pour the Stabilizer into the Saliva Vial, screw on the cap tightly, insert into the provided return mailer and mail.*** Do NOT provide your name or a return address. This is to protect your privacy.
4. ***Enter your nasal swab PCR test results at www.rta.biz/results/.***
This is a secure site. You will get the PCR results 3-10 days after your nasal swab. Fill in these two lines:

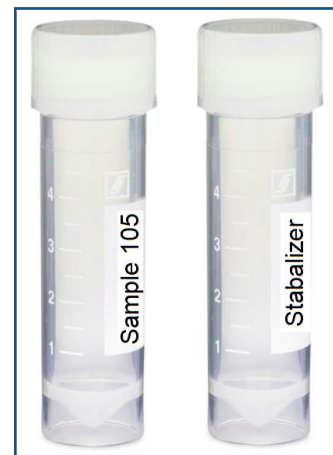

Saliva Sample number:
PCR Test Result: Positive (infected) Negative (not infected) PCR Ct number:
(if provided)

The Ct or C(t) number is very important to us. It will help us better define our sensitivity.

There is NO place to enter personal information.

We will measure your saliva sample as shown in the video (minus the saliva collection), and add the saliva and test results for your sample to our Performance Data.

And please check out our amazing team on this website.