


COVID-19 Saliva Antigen **At-Home** Test




Providing Chemical & Biological Information
When & Where You Need It

Test Kit Components in Pouch:

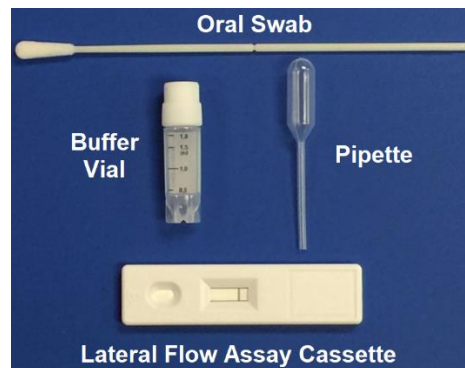


Test Instructions:


Store at 2-8 °C (35-45°F). Do not eat, drink or smoke 30 minutes before use, and drink water.

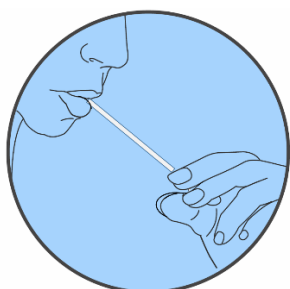


Place Swab on tongue & suck for **30 sec** to gather saliva on Swab.

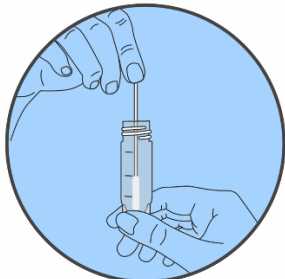


Insert Swab in Vial with Buffer & swirl for **1 min**.

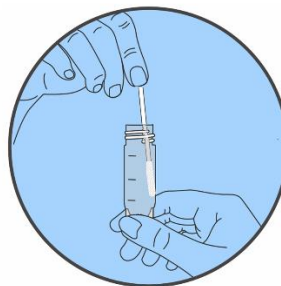


Press Swab against Vial walls to release solution.

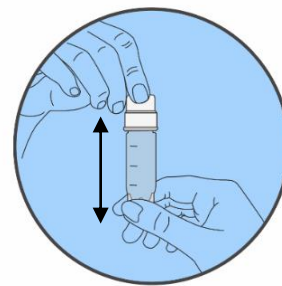


Remove Swab, Cap Vial & shake **5 times**.

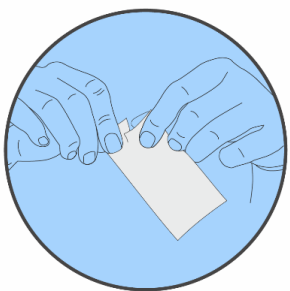

Remove Cassette from Pouch.

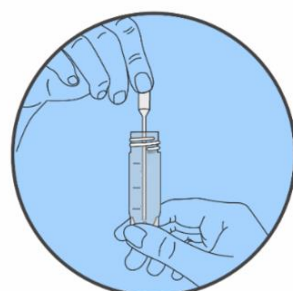

Draw sample into Pipette.

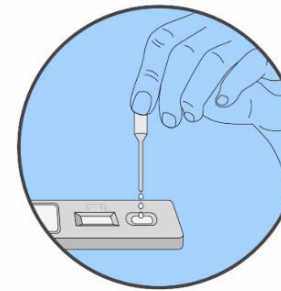


Add **3** drops of Sample to Cassette.




Wait **5-20 minutes** for 1 or 2 lines to appear.

Test Results:


Interpret Results within **20 minutes**. Test is invalid after **20 minutes** of adding sample.

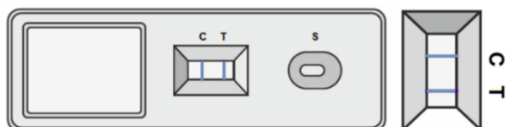

Blue-gray lines, **EVEN IF FAINT**, appearing at the T (Test) Line and the C (Control) Lines indicate a **POSITIVE TEST** and **LIKELY INFECTION**. **A follow-up PCR test is recommended.**

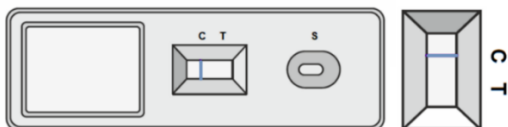

A single blue-grey line at the C (Control) Line indicates a **NEGATIVE TEST** and **LIKELY NO INFECTION**.

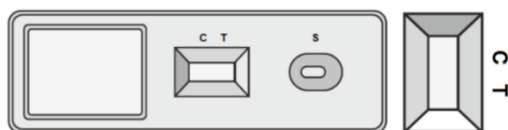

No lines at either the T (Test) or C (Control) lines indicate an **INVALID TEST**.

Warnings: Wear latex gloves & mask if testing others. Keep away from children. Reseal kit components in pouch & dispose of as Biohazard Waste.

Order Kits at www.RTA.biz